

# COOL•TIGHT

Love handles, Muffin top, back bulges, stomach, waist, hips, arms and calf

Time and cost savings by Covering 4 different areas at once  
with 4 channels or simultaneously treating two people.

Destroy stubborn fat with Cryopolysis ( $5^{\circ}\text{C} \sim -5^{\circ}\text{C}$ ), Activate fat releasing effect.  
Shrink and Relax muscle massage with Mid and Low Frequency



## AN INTEGRATE FAT CARE SYSTEM



### Contraction and Relaxation of muscle can be as beneficial as exercise and it makes muscle

Frequency adjustable from 150HZ to 750HZ that covers all body types. 400HZ is for normal sizes and lower the frequency for more obese person and higher the frequency for thin person. And also it can be adjustable per target depth of body from skin to deep layers for generating muscle.




### Accelerate frozen fat discharge

Medium and Low frequency therapy has similar effects which can get through exercising as exercising muscle with weak electrical signal and speed up metabolism without harm on the body.



### Reducing cold feeling

Locate low and mid frequency pad beside of Cryolipolysis applicators and work simultaneously with cryolipolysis function makes customers feel the cold less and feel stability with a gentle massage feeling.

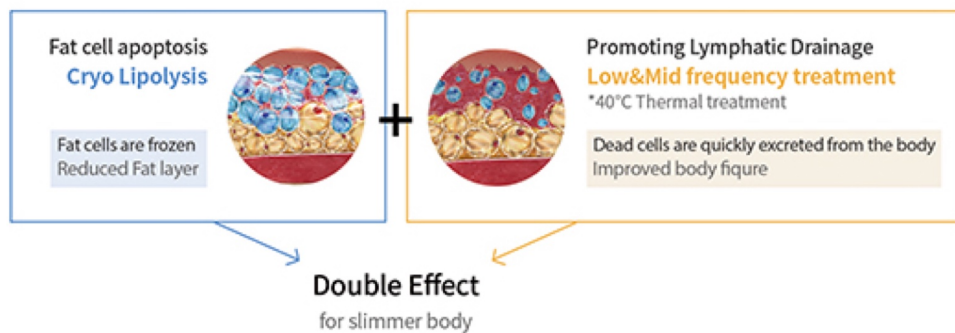

## No concern and complains about petechiae, bruising and after-pain

There is no suffering from the strong vacuum power because other devices make full contact with the skin with the freeze plates located on both inner sides of applicators.


To make contact skin with both sides of freeze plates, others apply very strong power to suck the skin into applicator and nerves are entrapped.

So complaints about petechiae, bruising and after-pain can be followed after treatment.


Our freeze plates are located on the upper side and gently suck in skin into applicator.

So patients can be treated without concern for pain caused by strong vacuum pulls of excess flab into the applicators.

tat

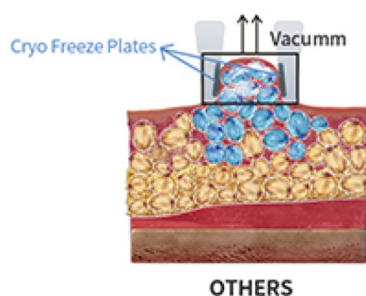

OTHERS

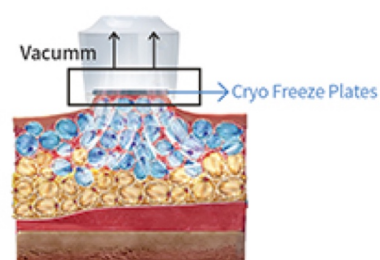


COOL•TIGHT



# PROVEN TECHNOLOGY AS NON INVASIVE FAT REMOVING, CRYO LIPOLYSIS

FAT Cells are frozen and dissolved.  
Once frozen, the cells dissolve and the released fat is likely processed by the liver.  
Clinical trials reported an average fat-layer reduction of 22.4 percent as measured by ultrasound.

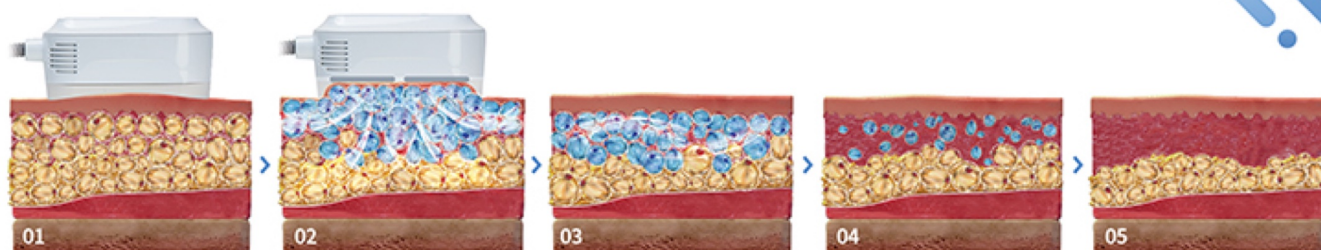


- 01 Apply H/P at the treatment area.
- 02 Suck in fatty area into applicators.
- 03 Cooling energy (5°C ~ -15°C) is applied, with the cooling energy, only fat cells are selectively cooled down.
- 04 Panniculitis is made in crystalized Lipocyte, they slowly die by apoptosis.
- 05 14 ~ 30 days after treatment Macrophages and other phagocytes surround, cover, and digest the lipid cells as part of the body's natural response to injury.  
2 to 3 months after treatment, Fat volume in the treated area is visibly decreased.

## Cryo Lipolysis

4 weeks interval, 40 min. to 1 hour 2 min.  
massage with hands upon completion of the treatment to improve the clinical outcome 10 min.  
Mid and Low frequency treatment.

## 4 Applicators

Shorten treatment time with 4 applicators at once.  
Save money by using one device for treating two people at once or covering 4 areas for one person.

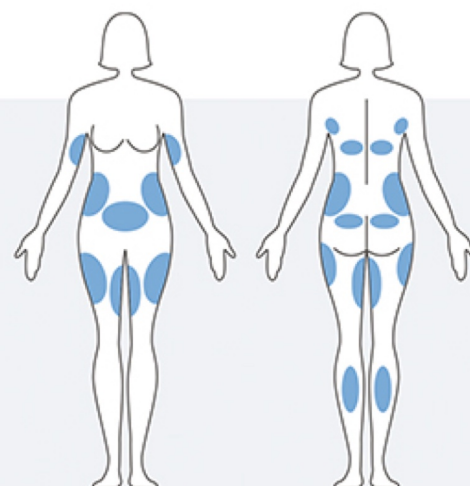





## — Compact design

Miracool has 2 different models. One has 4 applicators and the other one has 2 applicators.  
Both of them can be placed without space restriction.

## — Specification



### 4channels

**Power**

AC100 240V 50/60HZ

**Vacuum Range**

Maximum -350mmHg

Cooling temperature : -15°C ~ 5°C

Heat temperature : 35°C ~ 40°C

Mid and Low frequency : 250 ~ 700Hz

Dimension(WXDXH mm) : 40x32x89

LCD : 7 inch color touch LCD

Net Weight : 40kg

Max consumption : 360W



### 2channels

**Power**

AC100 240V 50/60HZ

**Vacuum Range**

Maximum -350mmHg

Cooling temperature : -10°C ~ 5°C

Heat temperature : 35°C ~ 40°C

Mid and Low frequency : 250 ~ 700Hz

Dimension(WXDXH mm) : 41x39x22

LCD : 7 inch color touch LCD

Net Weight : 10kg

Max consumption :180W

